



Ergo Attendant Control

Say goodbye to shaky joystick handling. The new Ergo Attendant Control lets caregivers guide the chair smoothly without the constant strain of fine joystick adjustments. The lever-based design stabilizes the hand and wrist, delivering intuitive navigation and a more confident driving experience.

Easy to Turn

The lever aligns perfectly with the push handle and the natural direction of the wheelchair, making wrist movement far easier and more stable than fingertip control.



Intuitive to Drive and Position

Driving is effortless: hold the lever with your fingertip to move forward, and gently push down to reverse. The same simple motion also makes it easy to adjust seating positions such as tilt and lift.



Modular Design

With a range of brackets, the Ergo Attendant Control system is fully compatible with VR2 Dual, R-net Dual, and R-net CJSM modules across six KARMA seating systems.

**The Ergo Attendant Control is compatible
with all KARMA power wheelchair seating systems**



Altus



Lectus



Versus



Sling



Kameleon



Ergo Stand



Seating	User	Attendant
Sling Versus Kameleon Ergo Stand	VR2, JSM	VR2, Dual Attendant
Kameleon	R-net, JSM	R-net, Dual Attendant
Sling Versus Kameleon Lectus Altus	R-net, CJSM	R-net, CJSM R-net, Dual Attendant

How to select attendant control

Feature	VR2 Dual	R-net Dual	R-net CJSM / CJSM2
Taking control / giving back	Push-button exchange between user & attendant	Push-button exchange between user & attendant	No handover button — control belongs to whoever powers on
Driver speed	Independent: attendant can set their own speed limit	Usually carries over from the user when swapped (depends on programming)	Independent: attendant can set their own speed limit
Ergonomics at push-handle	Compact Dual Attendant Module — light, easy to mount	Compact Dual Attendant Module — light, easy to mount	Bulky, heavier, with cables and screen inverted
On/Off button (attendant side)	No dedicated on/off	No dedicated on/off	Has on/off button
Seat/actuator controls	Yes — attendant can operate seating functions	Yes — attendant can operate seating functions	Yes — full seating menu, but more complex to navigate
Feedback to caregiver	Simple LED indicators	Simple LED/status indicators, Focus icon shows who is in control	Full LCD screen with status, modes, profiles

